Indications

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Indications

A metal dental bracket used in orthodontic treatment of malocclusions to correct the position and orientation of certain teeth.

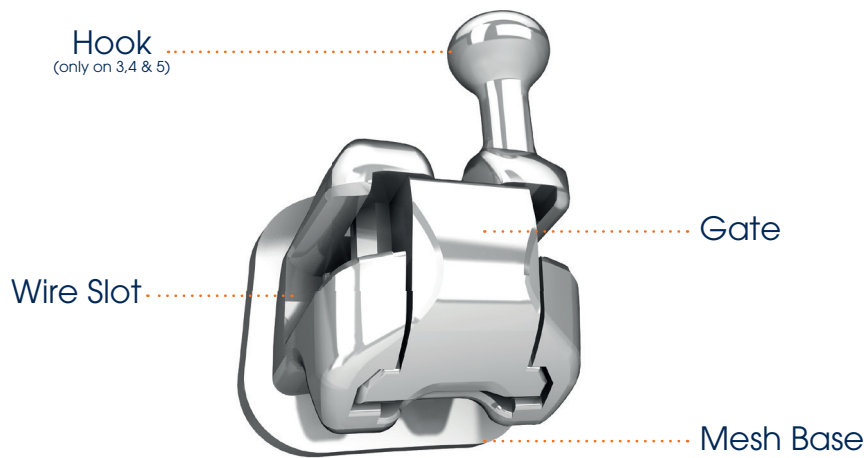
The bracket is bonded to the patient's tooth surface with an adhesive to apply pressure to the tooth via a flexible orthodontic wire.

The use of this medical device is restricted to orthodontic professionals. For single use only, this medical device is intended for adolescents and adults.

Delivered in non-sterile state, it is not intended for sterile use and must be disposed of after use in accordance with current waste disposal regulations.

Specificities of use

- Type of mounting of the Altitude SL® M bracket

- Opening and closing of the Altitude SL® M bracket

Open gate by pulling the front of bracket with usual dental explorer

Slightly Push the gate back forward by finger

Precautions for use

- Do not use chemicals, liquid H₂O₂ and disinfectants for cleaning and or soaking.
- Storage conditions: room temperature
- Do not expose this product nearby a heat source (< 55°C max).

Contraindications

Risk of allergies because the product contains nickel and chrome.

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