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**Ascend SL™ orthodontic brackets are bondable devices via a light-curing adhesive. They are composite based on a BisGMA resin system with a small portion of filler and a photosensitive catalyst system.**

## INDICATIONS

- Ascend SL™ orthodontic brackets are used for each patient with slightly and heavy malocclusion.
- They are intended to control movement of teeth to improve functional and aesthetic parameters of the stomatognathic system.
- They are intended to be bonded to patient's tooth with an adhesive in order to correct wrong tooth position during a period of 1 to max 2.5 years.
- The use of the medical devices is restricted to orthodontic professionals only.
- The medical devices are for single-use only. They are delivered in non-sterile state and should be cleaned with alcohol before being inserted into patient's mouth.
- Only NiTi Thermal wires are recommended to be mounted into bracket slot. The medical devices with light wires generate sliding mechanics that allow easy movement of the patient's teeth with low forces. This means quick results with less pain for the patient.


## PRECAUTIONS FOR USE

**CAUTION : These brackets should not be used on teeth with compromised enamel surfaces, on dental porcelain, or patients suffering from underlying gum disease. Bonding the bracket to compromised teeth can lead to tooth damage.**

- Clean and disinfect the brackets by soaking them in alcohol before use.
- Do not touch the brackets with fingers. Use the plier instead.
- The medical devices do not generally cause allergic reactions, but patients should consult their dentist in case an allergic reaction occurs.
- The brackets do not have an expiration date. After 7 years, we recommend to check the condition of change of the brackets and the damaged packaging.
- No special primers or pretreatments are necessary due to it is special bonding base. Orthodontic brackets can be used with traditional direct or indirect bonding methods. For exact bonding procedures, the recommendations of the adhesives manufacturer should be followed.

## STORAGE CONDITIONS

- Keep the brackets in their original packaging or in an airtight box in the dark at room temperature
- Do not expose the brackets directly into sunlight and UV lights.
- Although brackets withstand temperature up to 100°C, we recommend to store them with room temperature.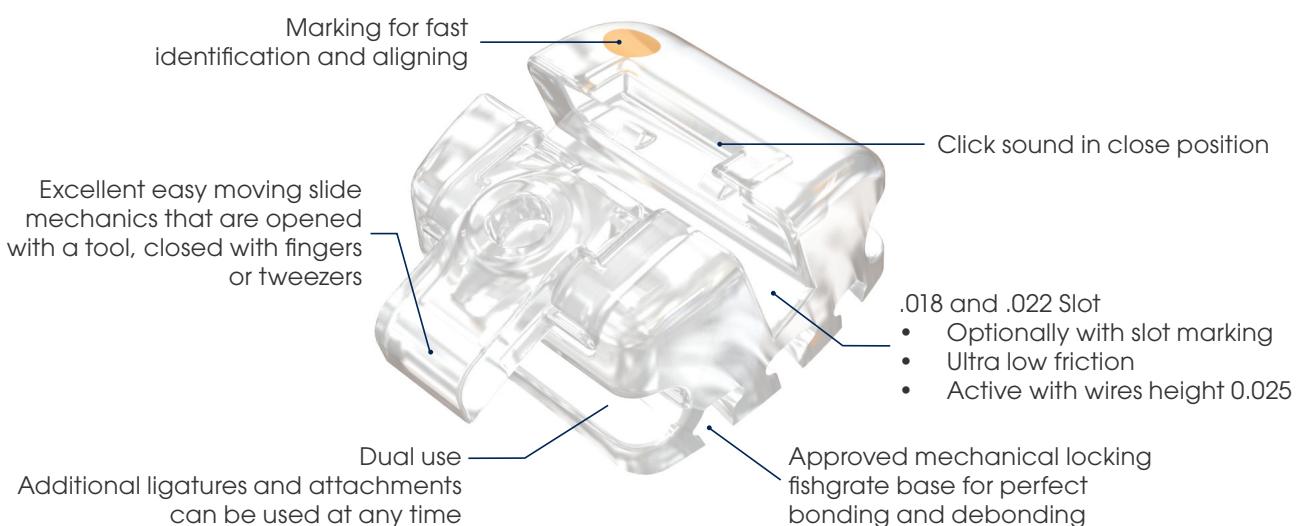

## INSTRUCTIONS FOR BONDING

Record the batch number on the label in the patient's file before removing the brackets from the package (necessary in case of complaint). Prepare the brackets by soaking them in alcohol to disinfect them. Do not touch them with your fingers and use the pliers or forceps. Prepare the teeth by cleaning and drying them carefully. Apply orthodontic adhesive on the bracket's base and attach it to the teeth. Remove adhesive flashes around the bracket carefully before curing with the light gun to minimize staining of the adhesive.

### 1 - Preparation

- Prepare the teeth with a non-fluoride and oil-free paste.
- Rinse with an oil-free air/water spray. Do not let the patient rinse.
- Isolate the tooth with a rubber dam or cotton rolls.
- Dry completely with an oil and moisture-free air source.

### 2 - Etching

- Etch the tooth according to the instructions of the etching material.

### 3 - Sealing

- Follow the instructions of the sealing material.

### 4 - Adhesion

- Keep the clip closed and clean the base of the bracket with alcohol before use. Do not touch the bracket with fingers.
- Apply a small amount of adhesive paste directly to the base of the bracket and make sure the paste fills all the spaces.
- Do not insert the adhesive paste into the bracket slot.
- Hold and fix the bracket with light pressure.
- Remove excess paste around the bracket.
- Use a curing light (470–480 nm, minimum 500 mW/cm<sup>2</sup>) and cure each bracket in the incisal-gingival or mesio-distal direction. The tip of the light guide should be within 5 mm from the tooth surface.
- The wires can be placed immediately after the last bracket is bonded. Remove any excess adhesive carefully.


## INSTRUCTIONS FOR DEBONDING

- Remove flash from around the brackets with a high-speed debonding bur.
- Use a bracket debonding instrument and place it in mesial and distal direction on the bracket base.
- Turn the bracket clockwise. In this way the bracket will shear off without breaking and the glue will mainly stay in the bracket base. This means that only a small portion of the adhesive will remain on the tooth after debonding.
- Remove excess adhesive with a high-speed debonding bur or adhesive removing plier.

## INSTRUCTIONS FOR OPENING/CLOSING


Position the orthodontic tool in front of the bracket and make sure that the pin is fitting exactly into the associated hole.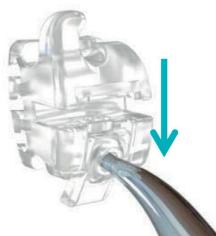

Pull the tool smoothly from the gingival to the occlusal side.

Insert wire.

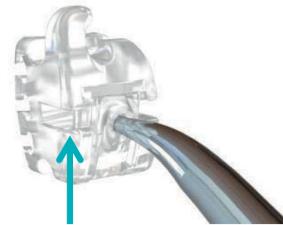After inserting the wire use the tool again to close the door. It is important to feel the door going over the hump and hear the click.

### Caution :

Do not use a standard dental probe. The end will be too flexible and will not close the door completely. This could damage it.

## RECOMMENDED WIRE SEQUENCES

**Caution : Only use NITI THERMAL WIRES**

Phase I	Aligning Stage	Deflection
<ul style="list-style-type: none"> <li>• Align and level</li> <li>• Initial arch form development</li> <li>• Initial derotation</li> <li>• Activating cellular reaction</li> <li>• Correcting malocclusion</li> </ul>	.013" Round Thermal NiTi .014" Round Thermal NiTi <b>Variations</b> .013" Round Thermal NiTi for severe Malocclusion .013" Round Thermal+ NiTi with push coil and not all teeth ligated	> 0.5 mm > 0.02"
Phase II	Levelling Stage	
<ul style="list-style-type: none"> <li>• Complete aligning and leveling</li> <li>• Complete rotations</li> <li>• Gain Torque control</li> <li>• Correcting angulations</li> <li>• Leveling Curve of Speed</li> </ul>	.014" Round Thermal NiTi .016" Round Thermal NiTi <b>For torque control</b> .014" x .016" Thermal NiTi <b>For highest torque control</b> .016" x .016" Thermal NiTi <b>For additional vertical leveling</b> .016" x .016" Super Elastic NiTi reversed curve	0.5mm÷0.35mm 0.020"÷ 0.014"
Phase III	Working Stage	
<ul style="list-style-type: none"> <li>• Closing of Spaces</li> <li>• Retracting anterior teeth with torque control</li> </ul>	.016" x .016" Thermal NiTi	< 0.35mm < 0.014"
Phase IV	Finishing Stage	
<ul style="list-style-type: none"> <li>• Correct midlines</li> <li>• Root alignment - finish torque control</li> <li>• Overbite / Overjet</li> <li>• Functional occlusal</li> </ul>	.016" x .016" Beta III Titanium 0.017" x .016" Thermal NiTi or SS 0.018" x .016" Thermal NiTi or SS	< 0.15mm < 0.006"
Phase V	Setting Stage	
<ul style="list-style-type: none"> <li>• Maximize intercuspidation</li> </ul>	.016" x .016" Thermal NiTi f Positioner	leveled

We applaud your decision to use Ascend SL aesthetic dental brackets to correct your malalignment. You are now wearing the most high-tech, next generation orthodontic appliance available. To keep them aesthetic and beautiful throughout the treatment, please follow the guidelines listed.

## CLEANING AFTER EATING:

While eating, food particles might get into the slot of the bracket, into the rails of the door and underneath the tie wings. Therefore it is necessary to rinse the teeth and the brackets with water after each meal.

To do so, just take a big sip of water and squeeze the water from the back in the mouth through the teeth to the front of the mouth. Then – with closed lips – move the water from left to right in the mouth. You may repeat this several times as needed.

## CLEANING WITH A TOOTHBRUSH:

Clean your teeth as usual with a toothbrush and toothpaste. It is more efficient to use an ultrasonic electrical tooth brush like Philips Sonicare Easy Clean in combination with a plaque removing toothpaste. This will help remove plaque that accumulates on and around the teeth and also on the brackets. This will help maintain the aesthetic look of the Ascend SL® bracket, as well as your teeth.

To make sure you remove all plaque that collects on the tooth surface as well as food particles on the brackets, we suggest the following cleaning technique:

- Brush vertically (up/down) several times to clean the front, left and right side of the brackets and teeth
- Brush horizontally with the toothbrush at a 45° angle from the gingival as well as the occlusal to clean the area underneath the tie wings and the wire.

## CLEANING WITH THE ORAL IRRIGATOR:


Additionally, we highly recommend to use oral irrigator. This irrigator shoots a water/air mixture through the bracket slot and through the door mechanism. In this way, all plaque and food debris are “blown” out. The use of mouthwash, instead of water, additionally helps protect against cavities and sensitive painful teeth.

## CLEANING AT THE ORTHODONTIST:

During the monthly visits with your orthodontist he/she will change the wire and therefore open the doors. The doctor can clean with a chair side water-air nozzle thoroughly, therefore ensuring that plaque and remaining food debris you couldn't remove before can be removed.

Twice a year, you should ask for a professional oral hygiene treatment to remove stains on the adhesive edges around the brackets. The doors should only be opened by the orthodontist. He/she has the correct instrument and knowledge to do so.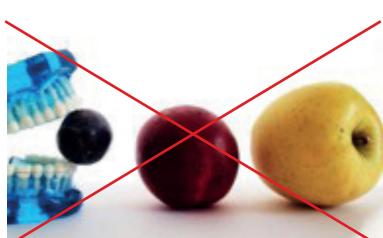

## OTHER USEFUL HINTS:

- **What to pay attention to when you're eating :** all food that requires biting hard, like some fruits, nuts, and bread should be chopped up before being eaten, otherwise the food might damage the brackets.
- Refrain from eating meat off the bone, chewing on pencils, fingernails, etc.
- Don't fiddle with toothpicks in and around the bracket.
- **What to pay attention to when drinking :** generally, you may drink anything you want. Be sure to rinse with water after drinking strong staining beverages like coke, coffee, red wine, etc.
- Smoking and chewing tobacco can also affect aesthetic brackets, so they should be avoided in effort to keep the brackets clean and maintain your health.

## REPROCESSING OF ORTHODONTIC INSTRUMENTS (used in treatment with the Ascend SL™ brackets)

### PRECAUTIONS FOR USE AND STORAGE

The current legal regulations in your country as well as the hygiene regulations of the orthodontist's clinic must be observed. Improper handling and maintenance, as well as improper use, can lead to premature wear of the orthodontic instruments in your possession.

The user is responsible for the sterility of the instruments and must ensure that validated cleaning, disinfection and sterilization processes are used.

- **Before each first use** : New orthodontic instruments must be thoroughly cleaned, disinfected and sterilized.
- **As a routine before each use :**
  - Examine each instrument thoroughly to ensure that it is not broken, cracked or defective, particularly at the level of the tips of the jaws, and pins.
  - Check the condition of the locking devices and moving parts.
  - Check that the size limit of the wire to be cut is respected, if applicable.
- **Protection of personnel and third parties** : Contaminated instruments should be disinfected as soon as possible during a treatment process to ensure maximum safety for the personnel handling them. When handling all used and contaminated instruments, it is necessary to wear protective gloves that comply with the requirements of Regulation (EU) 2016 / 425 on Personal Protective Equipment (PPE).
- **Cleaning, disinfection and sterilization process** : Only validated processes are used. Effective cleaning and disinfection are prerequisites for the correct sterilization of orthodontic instruments. In addition, the sterilization equipment must be regularly maintained and checked. The validated parameters for the cleaning, disinfection and sterilization cycles must also be monitored regularly.
- **Storage conditions** : In rooms with low humidity to avoid condensation.

**In the case of a return, please attach the "Proof of Decontamination" form to the instrument**

In compliance with the applicable statutory provisions and, above all, for the protection of our employees, we require a signed "Proof of decontamination" for each return. Please ensure that this "Proof of decontamination" is filled in and enclosed with each product return (complaint / repair / other reason for return) and package the product such that no risk of injury exists for our personnel in incoming goods inspection while unpacking.

Article no.: ..... Article designation: .....

If possible provide information on

Delivery note N°: ..... of [date] ..... Batch N°: .....

Reason for return: .....

We hereby confirm that

- The medical device(s) enclosed with the letter has/have not come into contact with blood, tissue or other bodily substances/fluids and hygienic safety can be confirmed by signing.
- The medical device(s) enclosed with the letter has/have come into contact with blood, tissue or other bodily substances/fluids and have been disinfected, cleaned and sterilized according to the applicable hygiene requirements for medical devices and the manufacturer's specifications, which we confirm by signing.

Specifications on disinfection, cleaning and sterilization:

- Disinfection and cleaning have been performed manually
- Disinfection and cleaning have been performed mechanically
- Steam sterilization (3 min. at 134°C / 15 min. at 121 °C)
- Other method (please specify)

The medical device(s) enclosed with the letter could not be decontaminated!

Reason: .....

Stamp from the institution

Date

Signature (and full name)

**IMPORTANT : Should we not receive this document or a comparable confirmation, we reserve the right to return the goods to the sender at their own expense.**

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