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Constat : Vis non vissée à fond.
Fact : The screw locking feature has not been activated.

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USAGE UNIQUE / THIS MEDICAL DEVICE IS FOR SINGLE USE :

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It is used in the elaboration of a unique appliance under medical prescription for a specific patient. The screw is self-locking, once de-mounted it must be replaced by a new one, in order to assure the locking feature. If re-used cross contamination can occur.

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
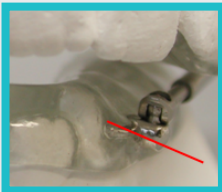
The pivot device is .020'' (5mm) long. It accepts an Elgiloy® wire from .045'' (1,1mm) to .047'' (1,2mm). The upper (maxillary) pivot device must be positioned at that the distal crown of first molar. The lower (mandibular) pivot device must be positioned at that the mesial crown level of first bicuspid. It is mandatory to respect the expected inclination of the supports.

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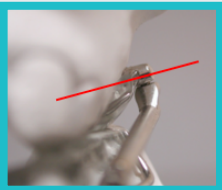The buccal acrylic must not interfere with the excursion movements (often at the bicuspid level). Take the time to remove the acrylic excess by using articulating paper. You can open buccal "windows" in the acrylic at the level of lower 1st & 2nd bicuspid. The appliance is less bulky and its resistance is not affected.

LIBERTY BIELLE™ SANS SURÉLÉVATION : LIBERTY BIELLE™ WITHOUT OCCLUSAL OVERLAP :

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The pivot device is .018'' (4,5mm) long. It accepts an Elgiloy® wire .036'' (0,9mm). The upper (maxillary) pivot device must be positioned at the distal crown level of the first molar. The lower (mandibular) pivot device must be positioned at that mesial crown level of the first bicuspid. The supports are parallel to the occlusal plane. With no occlusal plan the orientation of the support have less importance. Make sure the flat side of the pivot device (the side receiving the screw) is completely free of acrylic. If the wire has been laser welded to the pivot, it is important to bend at distance from welded (2-3mm), if not, a wire breakage may occur. If these tips are respected, the Liberty Bielle™ will give you full satisfaction.

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