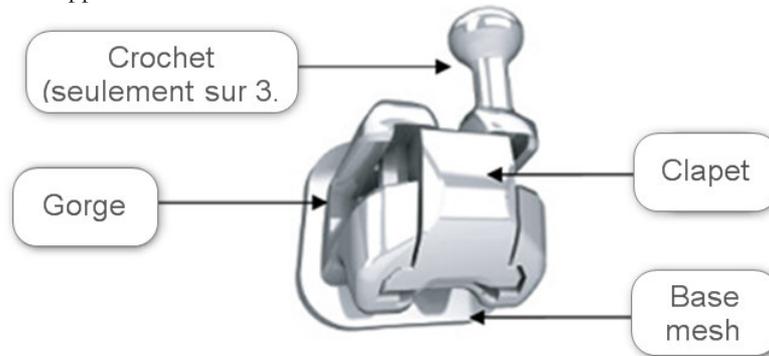
Altitude M SL

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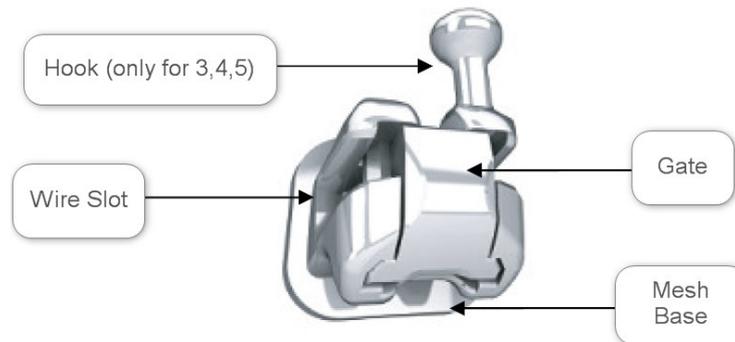
Altitude M SL

Indications for use:

The Dental Metal Bracket is intended for medical purposes to be bonded to a tooth to apply a pressure to a tooth from a flexible orthodontic wire to alter its position. The user population targets do not include newborn and infant.

- Safety Statement: This product is restricted for use by dental orthodontist.
- This product is intended for non-sterile use.
- The disposable of the product is to follow the local EPA regulations.
- Altitude M SL bracket instructions:

- Gate operation for Altitude M SL bracket:

Open gate by pulling the front of bracket with usual dental explorer

Slightly Push the gate back forward by finger

Precautions in using the device

Adverse effects : This product is prohibited to be exposed to high temperature.

Contraindications : The user population targets do not include newborn and infant.

Warnings : This product contains nickel and chromium and should not be used for individuals with known allergic sensitivity to these metals.

Transportation and storage conditions: Environmental temperature range : -40 °C ~ 55 °C

Atmospheric pressure: 500hPa – 1060hPa

Caution: This product is prohibited to be exposed to the chemical sterilization. This product is prohibited to be exposed to the use of H2O2 liquid or a disinfectant to clean, soak and wipe.

| Symbol | Description | Symbol | Description |
|--------|---|--------|---|
| | Indicates the date after which the medical device is not to be used. | | EC mark with Notified Body number |
| | Indicates a medical device that is intended for one use | | Manufacturer. |
| | Indicates the need for the user to consult the instructions for use. | | Indicates the manufacturer's batch code so that the batch or lot can be identified |
| | Indicates the manufacturer's catalog number. | | Indicates the Authorized Representative in the European Community. |
| | Indicates the date after which the medical device is not to be used. | | Indicates the temperature limits to which the medical device can be safely exposed. |
| | Caution: Indicates the need for the user to consult the instructions for use. | | |