J00040 Pasta J00041 Liquido

FUNDENTE LIQUIDO CON ACCION DETERGENTE

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DETERGENT-ACTION LIQUID FLUX

A special additive gives cleaner, better solder joints with all alloys and all types of soldering. The specially designed plastic dispensing bottle provides controlled flow (from the smallest to the largest solder need) by simple finger pressure. The self-clearing spout eliminates clogging and gives leak-proof closure.

SHAKE WELL BEFORE USING

For best results, always shake the container before using, since flux separates when idle. The special Formula 41 additive provides better cleaning action at the solder site. Squeeze the bottle, dispensing a single droplet. Light pressure controls the flow of flux. Keep elbow-action spout down (closed) when not in use. For special instructions on electric soldering, refer to RMO Equipment Manuals.

INSTRUCTIONS / WARNING

May be harmful to skin, eyes, and lungs. Avoid ingestion or contact with eyes, skin or clothing. Use only in well ventilated area. Do not breathe vapors. In case of contact with skin or eyes, flush thoroughly with water. If skin rash or eye irritation occurs, consult a physician. Keep container closed when not in use. Store between 60° F-80° F to insure maximum shelf life.

Tru-Chrome® Flux

J00040 Paste J00041 Liquid

Instructions for use

Attention,
See instructions
for use

RMO, Inc.

650 West Colfax Avenue
Denver, Colorado 80204 USA
To order call 303-592-8200
Toll Free 800-525-6375

European Representative RMO Europe

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