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Apply one thin layer of BIOPLAST® insulating agent only to those parts of the working model which will be covered by the mouthguard. Trim model flat and place it on the model platform with the clear V-shaped part of the BIOPLAST® insulating agent covering the desired extension of the anterior section (eg. from 3-3). Make sure the plate is fixed with the soft side down. Heat the plate by setting the code or recommended heating times according to the instructions. Lock pressure chamber and open after cooling phase has finished. Cut plate on model bottom.

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Warm up the occlusal areas under the infrared heater using the heating support.

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Cut mouthguard to the desired shape. Polish the edges with BIOPERM-Trimmer or DIMO®PRO wheel.

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Put the models into the articulator for counter bite. Cool down with compressed air or water and remove.

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Innovation for safety plus.
The new Mouthguard type **BIOPLAST® XTREME** gives safety plus in sports thanks to the integrated reinforcement in the delicate anterior region. Hard and flat impacts can be absorbed in contact sports, using this innovative new plate. **The innovation:** For the first time, different degrees of hardness are integrated in one pressure moulding plate. The size of the anterior protection area can be varied individually thanks to the V-shaped hard insert.

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Three Mouthguard types – meeting all requirements.

BIOPLAST® MASTER:
• proven mouthguard of two layers for maximum safety
• for all leisure time activities

The product innovations **BIOPLAST® XTREME** and **BIOPLAST® XTREME PRO** give safety plus thanks to specific extras.

BIOPLAST® XTREME:
• with integrated reinforcement segment in the anterior region
• for contact sports: absorbs hard and flat impacts

BIOPLAST® XTREME PRO:
• with integrated reinforcement segment plus hard-elastic layer in the anterior region
• for professional contact sports where hard and flat impacts often occur, absorbing the impact power over a large surface.

All mouthguard types as described above consist at least of 2 layers, guaranteeing maximum safety and comfort.

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Materialliste:
BIOPLAST® XTREME 5,0 mm
BIOPLAST®-Isoliermittel
Heiz-Support

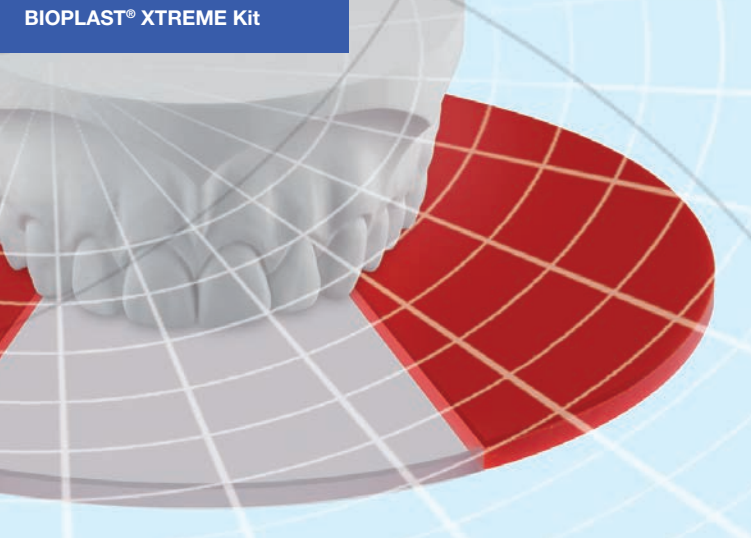
List of materials:
BIOPLAST® XTREME 5,0 mm
BIOPLAST® insulating agent
Heating Support

Liste des matériaux:
BIOPLAST® XTREME 5,0 mm
BIOPLAST® agent isolant
Support de chauffe

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
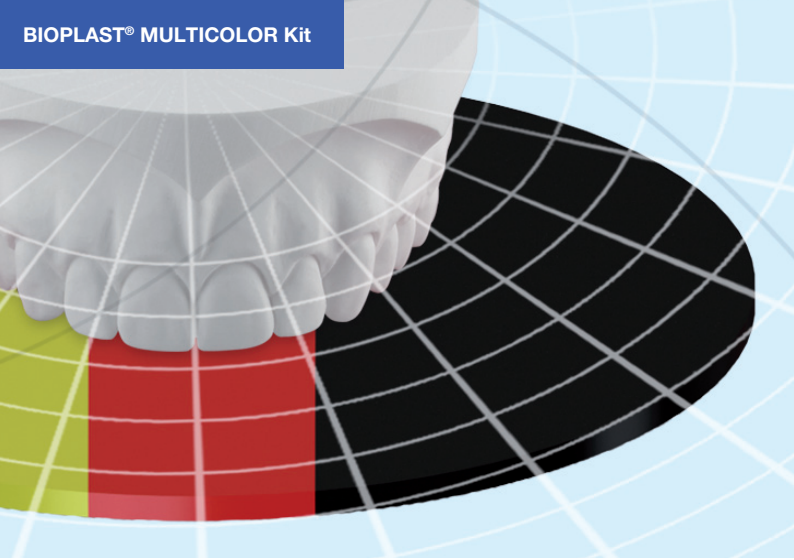
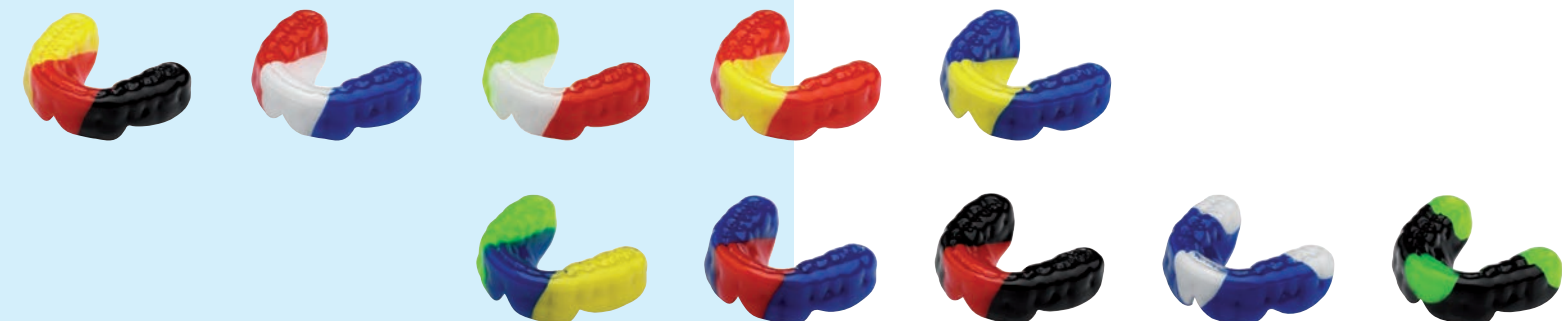
Attractive colors and various designs.

Individual mouthguard is the right complement of a professional protection for your leisure time activities as well as for professional competition. For your leisure time activities the **BIOPLAST® MASTER** is the proved and tested mouthguard, guaranteeing maximum safety.

The **BIOPLAST® MASTER** is available in 10 attractive colors and designs. Logos and names can be inserted, if required.

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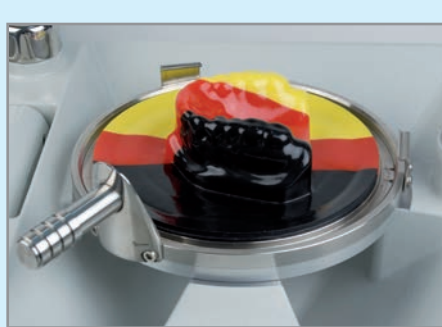
Apply a thin layer of BIOPLAST® insulating agent to the desired mouthguard area. Trim the model flat and place it on the model platform. Heat the plate by setting the code or recommended heating time according to the instructions. Lock pressure chamber and open after cooling phase has finished.

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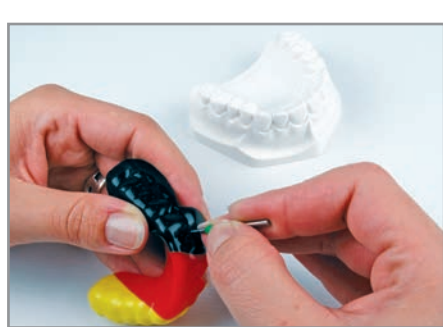Cut the 1st mouthguard plate with scissors or scalpel, polish the edges, clean with lukewarm water and let dry. If a reinforcement is desired, cut mesh layer according to jaw anatomy, heat occlusal surface using the heating support and adapt mesh layer. To avoid air inclusions, perforate plate interdentially at the papilla and occlusally with a drill bit. For optimum compound, make sure not to touch the surface again. Place model on the platform and heat 2nd plate by setting the code or recommended heating times. Lock pressure chamber and open after cooling phase has finished. Cut plate at model bottom.

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Warm up occlusal areas under the infra red heater and apply a thin layer of BIOPLAST® insulating agent occlusally to the antagonist model, in order to avoid plaster residues. Put the models into the articulator for counter bite. Cool down with compressed air and water and remove. Cut the mouthguard to the desired final form.

Polish the edges with BIOPERM-Trimmer or with DIMO®PRO wheel. Cut the mouthguard diagonally to remove sharp edges in the labial and palatal area. Finish the edges with the OSAMU polisher at a low RPM exerting low pressure.

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List of materials:

BIOPLAST® MULTICOLOR 3,0 mm or
 BIOPLAST® COLOR 3,0 mm
 BIOPLAST® clear 2,0 mm
 BIOPLAST® insulating agent
 Mesh layer
 Heating support

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BIOPLAST® MULTICOLOR 3,0 mm ou
 BIOPLAST® COLOR 3,0 mm
 BIOPLAST® clair 2,0 mm
 BIOPLAST® agent isolant
 Renfort de nylon
 Support de chauffe

Lista de materiales:

BIOPLAST® MULTICOLOR 3,0 mm o
 BIOPLAST® COLOR 3,0 mm
 BIOPLAST® transparente 2,0 mm
 BIOPLAST® líquido separador
 Malla de refuerzo
 Soporte de calentamiento