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Cheek retractor, saliva ejector and all connections to suction

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1. Work slide in channel several times before unit is used on patient.

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2. Place the saliva ejector in the mouth while at the same time inserting one of the "T" slides into the cheek retractor channel. Insert the other "T" slide.

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3. Have the patient place their tongue into the tongue guard and relax. When the patient is relaxed, the appliance will settle into a passive position.

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4. Slide the saliva ejector arms back until the cross over tubes are behind the last tooth to be bonded. Take some time to get the patient relaxed.

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5. Secure the suction hoses by placing them behind the retaining posts on the saliva ejector arms and on the cheek retractor spring.

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1. Bring the cheek retractors closer together and remove the entire unit while still connected together.

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2. After removal, do not leave the tongue guard connected to the cheek retractors. Otherwise, the cheek retractors will eventually pull the tongue guard apart.

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Autoclave (steam Sterilization) is recommended: Must not exceed 280° F (135°C). The entire Dry Field System can be safely autoclaved. The Silicone tubing, though, after repeated exposure to steam sterilization or long-term high temperature or pressure, will eventually relax and become gummy. It should then be replaced.

Helpful Tips:

1. Disconnect at least one arm of the tongue cage from the cheek retractor prior to sterilization. This relieves the spring tension on the tongue cage arms thus increasing the life expectancy of the component.

2. Make sure that you completely purge the entire system of saliva and debris prior sterilization.

3. Avoid arranging the system too close to other items in the autoclave as this will prevent steam from reaching all surfaces.

4. Prevent contact with metal instruments that are in the autoclave