DIRECTA **CONTAC**E**Z[®]**

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Composition

Plastic (polycarbonate)

Diamond plated and non-plated stainless steel

**Precaution**

- To be cleaned and disinfected according to the instructions below prior to first use and between uses. If high-level disinfection not possible or sterilization required by national recommendations, the device shall be sterilized according to the instructions.
- Use minimal vertical force to avoid breakage.

User instructions

Use two fingers and carefully pass the strip buccolingually, back and forth with gentle force.

Examples of use

ContactEZ Restorative Strip System: Proximal contact adjustment and complete marginal seating of crowns, veneers, inlays/onlays as well as proximal contouring of composite fillings.

ContactEZ IPR Strip System: Interproximal reduction system.

ContactEZ Sub Gingival Strip System: Removal of hard-to-reach subgingival rough margins and overhangs as well as final polishing.

ContactEZ Hygienist Strip: Removal of extrinsic discoloration and plaque from interproximal surfaces.

Maintenance, cleaning and sterilization

| | |
|-----------------------------|--|
| Product | ContactEZ Strips are used to remove and/or finish dental enamel or filling materials |
| Warnings | Place in the middle of the autoclave, not against the autoclave's walls or heating element |
| Limitations on reprocessing | The characteristics of the plastic are affected by autoclaving, resulting in limited product lifetime. Metal strip may discolor after autoclaving, does not affect safety or performance. |
| Inspection and maintenance | All plastic products have a limited-service lifetime and should be replaced at regular intervals. All types of cleaning reduce the product's service lifetime. The product should be replaced when its functioning and/or performance no longer satisfies the specifications in this manual. |

Instructions

| | | |
|-----------------------------|---------------|--|
| Preparation before cleaning | Instruction | Wipe off the instrument after use to prevent soil and debris to dry onto the instrument. Perform cleaning as soon as possible after use. Do not exceed 2 hours. |
| Cleaning: Manual | Equipment | Soft bristle brushes of various sizes, Ultrasonic bath |
| | Detergent | Follow agent manufacturers recommendations for concentration and temperature. Manual: Enzymatic or low-alkaline (pH≤8) detergent suitable for manual cleaning. Ultrasonic bath: Enzymatic or low-alkaline detergent with minimal foaming characteristics. |
| | Water quality | Drinking water quality for manual cleaning and distilled or de-mineralized water for ultrasonic bath and final rinse should be used. |
| | Instruction | <ol style="list-style-type: none">1. Immerse in freshly prepared cleaning solution as per manufacturer's instructions although maximum 40°C. Clean mechanically with a brush, working beneath the liquid level, until visibly clean.2. Rinse thoroughly3. Clean in an ultrasonic bath for a minimum of 5 minutes, using a frequency of 35-45 KHz and power min. 150 W. Make sure the instruments are in an open position.4. Rinse thoroughly for min 30 seconds.4 a Carefully dry instruments with clean compressed air (Class 1 or better according ISO 8573-1:2010) or lint free wipe. Any remaining water can cause potential rust or corrosion on the instrument surface and into the substrate.5. Inspect cleanliness. If necessary, repeat from step 1. |
| Disinfection: Manual | Detergent | Immerse into suitable disinfection solution. Follow the manufacturer's instruction of disinfection solution for concentration and time. |

| | | |
|---|--|---|
| Cleaning and disinfection: Automated | Equipment | Washer-disinfector (validated according to EN ISO 15883) Enzymatic or low alkaline, suited for medical devices. |
| | Detergent | Non-corrosive, neutral cleaning agent with minimal foaming characteristics, suited for medical devices. |
| | Rinsing Agent | Follow agent manufacturer's instructions regarding concentration and temperature. |
| | Water quality | Drinking water quality for cleaning and deionized or purified water for final rinse/ disinfection. |
| | Instructions | <ol style="list-style-type: none"> 1. Load the instruments in an open position in the washer-disinfector. Use a suitable instrument tray. 2. Run program: <ul style="list-style-type: none"> - Pre-wash in cold water, 2 x 2 min - Main wash with detergent at minimum 55°C, 10 min - Rinse in warm water, 2x 1 min. - Final rinse/disinfection in distilled or de-mineralized water at 90°C, minimum 1 min. - Drying at 110° C, minimum 15 min 3. When unloading, control that the instruments are clean. If necessary, repeat from step 1 or use manual cleaning |
| Inspection and maintenance | All plastic products have a limited-service lifetime and should be replaced at regular intervals. All types of cleaning reduce the product's service lifetime. The product should be replaced when its functioning and/or performance no longer satisfies the specifications in this manual. | |

| | | |
|-----------------------------|---|--|
| Packaging for sterilization | Equipment | Standard packaging pouch |
| | Instruction | Pack the instrument in an open position in the pouch. Place pouches plastic towards plastic and paper towards paper. |
| Sterilization | Caution | The instrument must be cleaned and disinfected before sterilization. |
| | Equipment | Steam autoclave (validated according to EN 13060, EN 285, EN ISO 17665-1) |
| | Instructions | Run minimal cycle: Steam temperature Min. 134 °C / 273°F Steam exposure time 4 min Vacuum drying 3 min |
| Storage | Ensure the instruments are dry before storage. The instrument should be stored in a packed tray, free from dust and moisture at moderate temperatures of 5°C - 40°C . | |
| Transportation | If required, secure transport in closed boxes is recommended in order to avoid any damage to the medical device as well as environmental contamination. | |

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Directa AB