Prior to use, carefully read the instructions for use.

ORTHO CONDITIONER

ORTHODONTIC CLEANING AGENT

For use only by a dental professional in the recommended indications.

INDICATIONS FOR USE

ORTHO CONDITIONER is a 10% polyacrylic acid solution designed to mildly etch and condition enamel, thus enhancing the bondability of glass ionomer to the tooth surface.

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to polyacrylic acid.

DIRECTIONS FOR USE

1. Clean tooth surface with pumice and water, prophy cup or brush. Rinse thoroughly and dry.
2. Dispense a drop of ORTHO CONDITIONER into a dappen dish or similar container.

Note:

1. After dispensing, remove any material remaining at the tip of the nozzle using wet gauze, etc. Close the bottle immediately after use.
3. Using a cotton pellet, apply ORTHO CONDITIONER for 20 seconds.
4. Rinse ORTHO CONDITIONER from the tooth. Keep the surface moist. The conditioned tooth is now ready for the application of GC Fuji ORTHO or GC Fuji ORTHO LC glass ionomer cement.

STORAGE

Recommended for optimal performance, store at temperature of 4-25°C (39.2-77.0°F).

PACKAGE

Bottle of 25 g (23.8 mL)

CAUTION

1. In case of contact with oral tissue or skin, flush immediately with water.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
4. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com> or for The Americas <http://www.gcamerica.com>

They can also be obtained from your supplier.

CLEANING AND DISINFECTING

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. **DO NOT IMMERGE.** Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

Undesired effects- Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM Ltd.
Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS, United Kingdom

Last revised: 09/2022

Rx Only

MANUFACTURED by
GC CORPORATION
76-1 Haasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan
RESPONSIBLE MANUFACTURER IN CANADA:
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
www.gcamerica.com

DISTRIBUTED by
GC CORPORATION
Rua Heliodora, 399, Santana - São Paulo, SP, BRASIL
76-1 Haasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900
www.gcamerica.com

GC SOUTH AMERICA
Rua Heliodora, 399, Santana - São Paulo, SP, BRASIL
CEP: 02022-051 - TEL: +55-11-2925-0965
CNPJ: 08.279.999/0001-61
RESP. TEC: Erick de Lima - CRO/SP 100.866

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