30000604 - NV9807

IFU Fuji I Capsule WEP

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Working Time (23°C/73°F) (min., sec.) 2'15"

Net Setting Time (37°C/99°F) (min., sec.) 2'30"

Finishing Time (37°C/99°F, after sealing the restoration) (min., sec.) 4'30"

Glas-Polyalkenoat-Zement, Luting, Netto Abbindezeit (37°C/99°F) (min., sec.) 2'30"

Glass-polyalkenoate cement, Luting, Net setting time (37°C/99°F) (min., sec.) 2'30"

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1. TOOTH PREPARATION

a) Prepare tooth in usual manner. For pulp capping, use calcium hydroxide.

b) Clean the prepared teeth with pumice and water.

c) Rinse thoroughly with water.

Remove excess moisture by blowing air or with a cotton pellet. gently blow off with oil-free air. DO NOT DESCALATE. Prepared surfaces should appear moist (glistening).

2. RESTORATION PREPARATION

Make sure that the restoration is pretreated and handled according to the manufacturer's instructions.

3. CAPSULE ACTIVATION AND MIXING

a) After activation, tap the capsule or tap its side on a hard surface to loosen the powder (Fig. 1).

b) To activate the capsule, use a plunger until the powder falls out with the body and hold it down for 2 seconds (Fig. 2).

Note:

1) Ensure the plunger is fully pressed to avoid the incorrect mixing ratio of powder and liquid.

2) The capsule should be activated just before mixing and used immediately.

3) Immediately set it into a mixer (or an amalgamator) and mix for 10 seconds (+/- 4,000 RPM) (Fig. 3).

Note:

When using a rotational mixing device, it is recommended not to use the centrifuge option.

4. DISINFECTION

a) Immediately remove the mixed capsule from the mixer and load it into the GC CAPSULE APPLIER.

b) Make sure of the nozzle, hold the applier with the capsule towards the nozzle (Fig. 4). To adjust the direction of the nozzle, hold the applier with the capsule towards you and turn the capsule body.

c) Extract directly into the restoration.

d) To remove the used capsule, push the applier release button.

5. CEMENTATION

a) Coat the internal surface of the restoration with sufficient cement and seat immediately. The working time is 2 minutes 15 seconds from the start of mixing at 23°C (73°F). Higher temperatures will shorten working time.

b) Maintain moderate pressure.

c) Stop removing excess cement when the excess cement feels rubbery.

d) Finishing can be started 4 minutes 30 seconds after sealing the restoration.

6. DISPOSAL

a) Coat the internal surface of the restoration with sufficient cement and seat immediately. The working time is 2 minutes 15 seconds from the start of mixing at 23°C (73°F). Higher temperatures will shorten working time.

b) Maintain moderate pressure.

c) Stop removing excess cement when the excess cement feels rubbery.

d) Finishing can be started 4 minutes 30 seconds after sealing the restoration.

7. STORAGE

Recommended for optimal performance, store the capsules in the original aluminum foil in a cool and dark place (4-25°C) (39.2-77.0°F).

8. SHADE

Light yellow.

9. PACKAGE

50 capsules

Average contents per capsule : 0.33g powder and 0.18g (0.14mL) liquid. Minimum net volume of mixed cement per capsule : 0.19mL.

Option : GC CAPSULE APPLIER (1 piece).

10. CAUTION

1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water. To avoid the spread of infection, do not touch the oral tissue.

2. In case of contact with eyes, flush immediately with water and seek medical attention.

3. This product is not intended for nail or core build-up.

4. Personal protection equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

5. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and consult a physician.

Some products referred to in the present IFU may be classified as hazardous according GHS.

Always familiarize yourself with the safety data sheets available at: <https://www.gc.dental/europe>

For the Summary of Safety and Clinical Performance (SSCP) please see EUADAMED database (<https://ec.europa.eu/tools/euadamed>) or contact us at [Regulatory@gc.dental](mailto:Regulatory@gc.dental).

Undesired effects - Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instructions, please report them to the relevant authority of your country accessible through the following link:

[https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en)

vigilance@gc.dental

In this way you will contribute to improve the safety of this product.

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