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# TRU-CHROME PASTE FLUX - PASTE FLUX FOR FLAME SOLDERING / J00042

## INDICATIONS :

**TRU-CHROME PASTE FLUX** is a special additive for making all types of clean and tight welds with all types of alloys. The product is exclusively intended for use by health professionals who must have sufficient knowledge to guarantee its correct handling.

**Each health care professional is required to read and carefully observe the safety precautions when handling the product and the necessary preventive/protective measures described in this document.**

## STORAGE & DISPOSAL CONDITIONS :

The container must be kept tightly closed and stored in a clean, well-ventilated place. After opening, the expiration date of the product is extended to 1 year.

The product must NOT be discharged into the environment; its disposal must be carried out through an authorized site.

## PRECAUTIONS FOR USE :

**Caution : This product may be harmful to the skin, eyes and lungs. It can cause severe skin burns, eye damage and eye irritation. It may also harm fertility or the unborn child.**

### Tips for each use :

- Make sure the product is sealed before opening.
- Ensure proper ventilation of the workstation.
- Avoid breathing vapors.
- Wear gloves, appropriate clothing, and eye and face protection.
- Do not eat, drink or smoke while using this product.
- Always wash your hands after handling the product.

**Contraindications :** The product is incompatible with alkaline earth metals.

## SECURITY INFORMATION :

**Caution : Contact a poison control center or doctor immediately in case of incident.**

- **If exposed or concerned :** get medical advice/guidance and call the poison control center immediately.
- **If inhaled :** Remove to fresh air and keep person comfortable to breathe.
- **In case of skin and/or hair contact :** immediately remove all contaminated clothing and flush skin with plenty of water.
- **In case of contact with eyes :** rinse cautiously with water for several minutes; remove contact lenses if necessary and rinse eyes with plenty of water.
- **If swallowed :** rinse mouth; DO NOT induce vomiting; call poison control center/physician if you feel unwell.

### IMPORTANT

**Any liability for products that have been modified or used for purposes other than those intended or in an inappropriate manner is excluded.**

**Any product-related adverse reaction should be reported to the manufacturer and any serious incident should be reported without delay to the competent authority of the Member State in which the patients and practitioners are established.**

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