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Cleaning and Reprocessing Instructions Multi – Family Appliances

At home by patients

Inspect your appliance before placing it in your mouth. If you detect signs of abnormal wear or damage, discontinue use and contact your dentist.

Rinse the appliance daily before placing it in your mouth and after removing it from your mouth.

Use a toothbrush to clean the appliance. Do not use toothpaste because abrasive ingredients in toothpastes may scratch your appliance. The appliance can be disinfected if necessary by immersion in boiling water for at least 1 minute. The box can be washed in a dishwasher.

By professionals

Adhere to guidelines for infection control in dental health-care settings in accordance with local regulations and recommendations.

Follow the instructions for use provided by the manufacturer of the washer-disinfector, autoclave, detergent and/or sterilant.

As the appliance comes into contact with mucous membranes, it is considered a semi-critical item.

The appliance is intended to be used on one patient and then discarded. It may exceptionally be reprocessed (inspected, cleaned and high-level disinfected or sterilized) for use on another patient in case the appliance has been used on a same-day procedure (e.g. fitting the appliance) under the monitoring of health care professionals.

If the appliance has been used by a patient at home, it shall not be reprocessed for use on another patient.

Inspection: Inspect the appliance visually and discard if any signs of damage are detected.

Cleaning : Clean the appliance without delay. Do not presoak or use chemical disinfectants.

Recommendations : Washer-disinfector:

Clean in washer-disinfector using a mild pH-neutral enzymatic detergent that is intended for cleaning of medical devices made of silicone and that is not classified as a dangerous preparation under directive 1999/45/EC or other local regulations.(Check “Section 2: Hazards identification” of the material safety data sheet provided by the detergent manufacturer.)

To avoid air bubbles and to ensure proper cleaning, rinsing and disinfection of the holes, place the appliance in an upright position with the holes vertically. The temperature of the washer-disinfector program can be freely chosen as silicone is heat tolerant (134°C).

Recommendations : Manual cleaning :

Scrub with a soft brush using a mild pH-neutral enzymatic detergent that is intended for cleaning of medical devices made of silicone and that is not classified as a dangerous preparation under directive 1999/45/EC or other local regulations. (Check “Section 2: Hazards identification” of the material safety data sheet provided by the detergent manufacturer.)

Rinse thoroughly. After cleaning, inspect the appliance and repeat the above if necessary. Make sure the appliance and its holes are thoroughly cleaned and rinsed.

Recommendations : Sterilization :

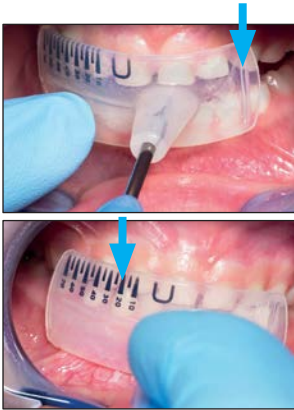
Sterilization in steam autoclave at 134°C.

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www.rmoeurope.com

Selecting size

Place the elevated marker between the left maxillary lateral incisor and canine.

Read the scale between the right maxillary lateral incisor and canine. Sizes 10, 20, 30... are indicated with larger markers whereas sizes 15, 25, 35... are indicated with smaller markers.

In case of crowding or diastemas, consider a larger or smaller appliance. Fit Multi-P Sizer on the patient and double check that the teeth fits properly in the slots.

Appliances used can be sterilized in an autoclave.

Fitting

Fitting is the most important step of size selection

Fig. 1 The canine rests at the bottom of its slot and Multi-P Sizer places no mesiodistal-force on the canine. If there is no crowding or risk of crowding, the size in the photo is correct. However, if crowding is present or expected, a larger size should be selected to enable expansion of the arch perimeter.

Fig. 2 One size larger than in Fig.1. The canine is guided by Multi-P Sizer towards the bottom of the canine's slot and applies a force that pushes the canine distally. This size is appropriate if crowding is present or expected and arch perimeter expansion is needed.

Fig. 3 Three sizes larger than in Fig. 1. The appliance is too large. The canine is guided against a ridge between two slots and the appliance does not guide teeth properly. Choose a smaller size. Sterilize Multi-P Sizer in the autoclave.